



Model 7469 LARGE PET DIGITAL SCALE by **REDMON**



SEE THE REDMON COMPANY 125 year HISTORY BY VISITING EITHER OF OUR WEBSITES

www.RedmonUSA.com

www.UVZHealthSystems.com

OTHER POPULAR BRANDS OF THE REDMON COMPANY:



REDMON

FEATURES:

- **Large Digital Readout.**
- **Large platform size (26" x 15").**
- **Non-skid rubber textured mat.**
- **Safe no-tip design.**
- **Requires 2 Lithium batteries (included).**
- **Capacity 8 oz. to 220 lbs (0.03kg/1 oz graduation).**
- **Tap on feature.**
- **Auto off feature (after 15 consecutive seconds of non-use).**

PREPARATION FOR USE:

1. You must remove the battery insulating strip before operating. Open the battery cover located on the bottom of the scale. Be sure to put the battery cover back once the battery insulating strip is removed.
2. Use the scale only on level, hard surfaces. Do not use on carpet.

USE:

1. Set the unit of measure switch (kg/lbs) to the desired unit (bottom of scale inside the battery compartment).
2. The scale does not have an on/off switch. Place your pet on the scale (reasonably centered). Your scale will automatically activate. Note the weight and record on the Weight Monitoring Chart.
3. The display will automatically turn off after 15 seconds of non-use.

CARE:

1. Use and store only in dry area.
2. Do not expose to extreme temperature and/or high humidity (best operating temp 65° – 85°).
3. Do not overload scale beyond the maximum weight (220 lb).
4. Do not use directly beside microwave oven or similar microwave transmitters.

